DEUTSCH

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ENGLISH

FOR OVERLAYING CULTURES OF GAMETS, ZYGOTS OR EMBRYOS

INDICATION FOR USE

VOIL is intended for use as an overlay for culture of gametes, zygotes, or embryos in Assisted Reproduction Technology (ART) and micro-manipulation procedure.

PRODUCT DESCRIPTION

VOIL is a ready-to-use pre-washed, light paraffin oil for use as an overlay for culture of gametes, zygotes, or embryos in Assisted Reproduction Technology (ART) and micro-manipulation procedure to prevent the evaporation of water and protects the media against changes in osmolality and pH. Culture of embryos in small volumes of medium under oil helps to enhance rates of embryonic development.

COMPOSITION

- mineral (paraffin) oil

PRODUCT SPECIFICATIONS

- Endotoxin (LAL): < 0.1 EU/mL
- Sterility: sterile (no growth)
- Density: 0.83 - 0.86 g/mL
- Viscosity: < 30 cP at 30 °C
- Peroxide Value (POV): < 0.1 mEq/kg
- MEA 1-cell to expanded blastocyst in 120h: ≥ 80%
- A certificate of analysis and MSDS are available upon request.

STERILE A Sterilized using aseptic processing techniques.

CONTRAINDICATION

No known contraindication.

WARNING

- Only for the intended use.
- Do not use if the package is damaged.
- Not sterile after opening.
- Do not resterilize.
- Do not reuse.
- Multiple use can lead to cross contamination.
- Do not use if there are signs of microbial contamination.
- Do not use if seal of the container is damaged upon delivery.
- Do not freeze before use.

PRECAUTIONS

- Rx Only** Caution: Federal law (U.S.) restricts this device to sale by or on the order of a physician.
- VITROMED recommends handling media only with aseptic techniques (e.g. in a laminar flow workbench, ISO class 5).
- Leave product in its original packaging.
- The products may only be disposed of in accordance with the applicable regulations for waste products.
- Do not use any medium that shows particles, turbidity or discoloration.
- VOIL may appear cloudy when removed from the refrigerator or at room temperature after extended periods of storage time. Such cloudiness usually disappears when the oil is equilibrated in the incubator for several hours. If it does not disappear discard the product.

- Minimal amounts of excess media in the bottle must be discarded.
- Any human, organic material is considered potentially infectious. All samples should be handled as if they could transmit HIV or hepatitis.
- Always wear protective clothing when handling samples.

INSTRUCTIONS FOR USE

VOIL should be used only for its intended purpose by qualified and trained persons. Each laboratory should consult its own optimized and validated laboratory procedures.

- VOIL is pre-washed, no further preparation is necessary.
- VITROMED recommends preparing dishes/flasks with oil the day before use.
- Incubate dishes/flasks at 37°C in a CO₂ incubator so that the oil warms up and becomes saturated with gas.
- VITROMED recommends incubation of aliquots. If flasks are incubated, be sure not to tighten the cap of the flask because this will prevent gas exchange.
- Use a sterile, cell-culture suited pipette to add culture medium to a vessel which ensures gas exchange with the CO₂ atmosphere.
- Add an amount of oil sufficient to cover the whole medium surface.
- VOIL is compatible with all VITROMEDIA products.

STORAGE INSTRUCTIONS AND STABILITY

- Do not use after the expiration date.
- Store VOIL between 2 °C to 8 °C.
- Keep away from sunlight.
- VOIL can be used safely for at least 28 days after opening the container, if it is handled under aseptic conditions.

FEEDBACK AND OBLIGATION TO INFORM

Thank you for purchasing our product. If you have any suggestions for us to improve our products, please send them to qc@vitromed.com. Report any serious incidents that have occurred with the product immediately by email to qc@vitromed.com and the competent authority of the Member State in which the user and/or patient is established.

SYMBOL GLOSSARY
(ISO 15223-1:2016 Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements)

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Technischer Support
Technical support

VITROMED GmbH
Raiffeisenstr. 15a
40764 Langenfeld
Germany

+ 49 2173 . 200 41 30
support@vitromed.com
www.vitromed.com

Stand der Informationen / Date of information
06-Feb-2020