VICSIPET

DEUTSCH

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VICSIPET

ENGLISH

VICSIPET PIPETTES FOR THE MANIPULATION OF CELLS WITH THE HELP OF A MICROMANIPULATOR.

INDICATIONS FOR USE

VICSIPET-TROPHECTODERM BIOPSY

For the collection of trophoctoderm cells for subsequent preimplantation genetic diagnosis.

VICSIPET-BLASTOMERE BIOPSY

For the collection of blastomeres for subsequent preimplantation genetic diagnosis.

PRODUCT DESCRIPTION

VICSIPET is a borosilicate glass micropipette for use in assisted reproduction. The pipettes are available with different tips, inner diameters and angles.

PRODUCT SPECIFICATIONS

- Material: Borosilicate glass, non-pyrogenic
- Primary packaging: Polypropylene
- Secondary packaging: Cellulose
- Endotoxins: < 0.05 EU/device
- Sterility: sterile (SAL 10⁻⁶)
- MEA 1-Cell to expanded blastocyst in 96h: ≥80%
- A certificate of analysis and MSDS are available upon request.

STERILE R Sterilized by gamma irradiation.

CONTRAINDICATION

No known contraindication.

WARNINGS

- Only for intended use.
- Do not use if package is damaged.
- Not sterile after opening.
- Do not re-sterilize.
- Do not reuse.
- Multiple use may lead to in cross-contamination.
- Each laboratory should determine which VICSIPET variants and angles are to be used for the specific applications. Using VICSIPET INJECTION with spike makes it easier to penetrate and causes less damage to the oocyte.

PRECAUTIONS

- VITROMED recommends handling under aseptic conditions.
- Carefully inspect the pipette after removing it from the pack and ensure that there is no damage. Do not use broken pipettes.
- The products may only be disposed of in accordance with the applicable regulations for waste products.

Rx Only Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

INSTRUCTIONS FOR USE

VICSIPET should only be used for its intended purpose by staff trained with the procedure.

- ① Hold the VICSIPET box on the right and left sides at the lateral indentations using your thumb and middle finger. The VITROMED logo and small bump on the long side of the box is facing you.
- ② Gently apply pressure on the small bump on the front, the shutter is unlocked and the box is opened. Flip open the lid completely.
- ③ VICSIPET is fixed by two holders. Grasp the pipette exactly between those holders and then lift the pipette gently and straight up out of the holders.
WARNING: Avoid pushing the pipette forward or back in its holder, because this may damage it!
- ④ Insert the pipette into the pipette holder of the micromanipulator and mount it according to the corresponding operating instructions.
- ⑤ Use the VICSIPET according to the indications for use.
- ⑥ Correctly dispose VICSIPET after use.

STORAGE INSTRUCTIONS AND STABILITY

- Do not use after expiry date.
- Keep away from sunlight and heat.
- Keep dry.

FEEDBACK AND OBLIGATION TO INFORM

Thank you for purchasing our product. If you have any suggestions for us to improve our products, please send them to qc@vitromed.com. Report any serious incidents that have occurred with the product immediately by email to qc@vitromed.com and the competent authority of the Member State in which the user and/or patient is established.

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VITROMED GmbH