V^{CARE} INCUCHECK

ENGLISH

LIQUID MEDIA CONTAINING PHENOL RED FOR CONTROL OF PH FOLLOWING INCUBATION IN CO₂-INCUBATORS

INDICATIONS FOR USE

V^{CARE} INCUCHECK is intended for the control of correct CO₂-settings of incubators by controlling the resulting pH. It is an additional control beside pH control of the culture medium, that can be conducted more frequently and in higher volumes to ensure more process safety.

Even if a blood blood-gas analyser is used that principally enables small volume pH-measurements, incubation of larger volumes can be highly beneficial. In contrast to low volume incubations of culture medium, degassing of larger volumes is negligible before the medium is injected in the analyser.

CONTRAINDICATION

No known contraindications.

PRODUCT DESCRIPTION

V^{CARE} INCUCHECK is a ready-to-use medium, which contains phenol red. V^{CARE} INCUCHECK does not contain growth substrate which, in combination with Gentamicin, prevents most microbial growth effectively for over a week even when handled unsterile.

COMPOSITION

- Ultra-Pure Water
- Gentamicin
- Bicarbonate
- Phenol Red

PRODUCT SPECIFICATIONS

- Sterility: sterile (no growth)
- pH: 7.20 – 7.40 (measured at 6% CO₂ at sea level)
- Visual control: passed
- A certificate of analysis and MSDs are available upon request.

WARNING

- Do not use after expiry date.
- Do not re-sterilize after opening or reuse by putting used medium back the bottle
- Do not use the product if any signs of microbial contamination, like turbidity, discoloration or a precipitate is observed.
- Do not use if seal of the container is damaged upon delivery.
- V^{CARE} INCUCHECK contains gentamicin, appropriate Precautions should be taken to ensure that the operator is not sensitized to this antibiotic (allergic reaction).
- Using V^{CARE} INCUCHECK does not substitute measuring of pH of embryo culture medium.

PRECAUTIONS

- Always wear protective clothing when handling samples.
- The product may only be disposed of in accordance with the applicable regulations for waste products.

INSTRUCTIONS FOR USE

V^{CARE} INCUCHECK should be used only for its intended purpose by qualified and trained persons. VITROMED recommends that each laboratory uses its own optimized and validated laboratory procedures.

pH control in a moist CO₂ incubator

1. Open the bottle V^{CARE} INCUCHECK.

2. Remove the desired volume of product in the container that you will use for pH measurement. We recommend to use at least 10 mL to avoid degassing while moving the product.
3. If available, a suitable pH probe that measures pH inside the incubator (e.g. Bluetooth connected) is added to the sample before incubation.
4. Incubate it in the moist CO₂ incubator overnight to 24h prior to measurement.
5. Use your own optimized and validated laboratory procedures for pH measurement.
6. Consult also pH measurement results of the embryo culture medium and, if needed, adjust the CO₂-level of the incubator.
7. We recommend to discard bottles opened under sterile conditions after one month.

The product can remain inside the incubator for up to 7 days after opening/pH-measurement.

pH control in a dry CO₂ incubator

Follow the steps as described before but conduct the incubation with oil-overlay or with an additional container of water with a large surface. Oil can disturb pH-electrodes, contact should be avoided.

Method 1: remove the oil overlay just before measurement.

Method 2: carefully remove V^{CARE} INCUCHECK with a pipet under the oil layer and pipet in a new container with as less turbulences as possible.

Method 3: Place a suitable pH probe in V^{CARE} INCUCHECK before adding the oil layer.

STORAGE INSTRUCTIONS AND STABILITY

- Do not use after expiry date.
- Store between 2°C to 37°C.
- The product can be used for a period of up to 30 days after opening, provided that it is handled under aseptic conditions.
- Keep away from sunlight.

FEEDBACK AND OBLIGATION TO INFORM

Thank you for purchasing our product. If you have any suggestions for us to improve our products, please send them to qc@vitromed.com.

Report any incidents that have occurred with the product immediately by email to qc@vitromed.com.

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VITROMED GmbH